Medicom SAS
 Boulevard de la chanterie | 49124 Saint Barthélemy d'Anjou
 France
 Tel +33 0 241 963 434 | sales.support.pro.eu@medicom.com | www.medicom-eu.com

**Notice d'utilisation / Instructions for use / Instrucciones de uso / Gebrauchsanweisung / Istruzioni per l'uso /
 Instruções de utilização / Gebruiksaanwijzing**

FRANÇAIS – FR

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ENGLISH – EN

Category I personal protective shoe covers (Regulation EU 2016/425), designed to provide protection against minor risks only (small splashes of cleaning materials of weak action or prolonged contact with water and superficial mechanical injuries), and to limit contamination in sensitive areas.

Reference	Size	Colour
10.001D-10R	L	White
H01038AR	L	White
10.040R	XL	White with blue sole
10.041R	XL	White with white sole

Limitations on use: The user will be the sole judge of the suitability of the protective accessories for the situation and the protection required. The manufacturer cannot be held responsible for the improper or inappropriate use of the accessories. The shoe covers are for single use only and must be disposed after any job, are for professional use and do not protect against biological and chemical hazards. Do not use them if any defects are noticed (e.g.: holes, damaged seams, ...). The shoe covers are flammable, keep away from fire.

Performances: The shoe covers meet the requirements according to the EN ISO 13688:2013+A1:2021 and the following standards:

Tests	Standards	Results
Tests on the nonwoven		
Resistance to water penetration – increasing hydrostatic pressure	EN ISO 811 :2018	13.4 cm H ₂ O
Abrasion resistance	EN ISO 12947-2 : 2017	6 000 cycles
Tensile strength	EN ISO 9073-3 :2023	Machine direction : 70 N Sens travers : 55 N
Elongation at maximum force	EN ISO 9073-3 :2023	Machine direction : 47 N Cross direction : 55 N
pH	EN ISO 3071 :2020	6.0
Tests on the sole		
Resistance to water penetration – increasing hydrostatic pressure	EN ISO 811 :2018	> 2 000 cm H ₂ O
Abrasion resistance	EN ISO 12947-2 : 2017	45 000 cycles
Tensile strength	EN ISO 9073-3 :2023	Machine direction : 134 N Cross direction : 93 N
Elongation at maximum force	EN ISO 9073-3 :2023	Machine direction : 63 N Cross direction : 66 N

Definition of traceability pictograms:

 Date of manufacturing  Expiry date  Batch number

Shelf life: Use the product within 5 years of its date of manufacture.

Storage and disposal: Shoe covers should be stored in their original packaging in a dry place and away from sunlight. They must be disposed of in accordance with the regulations in force.

The internet address at which the declaration of conformity can be accessed is: www.medicom-eu.com

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