Swiss Authorised Representative (CH-REP)
FINAGEST SA, Rue Pierre-Fatio 12, 1204 Genève - Switzerland
Swiss Single Registration Number (CHRN): CHRN-AR-20002401

EN Reusable face drapes
FR Champs faciaux réutilisables
DE Wiederverwendbare Gesichtstücher
ES Cortinas reutilizables
IT Teli riutilizzabili per il viso
PL Chusta do osłony twarzy wielokrotnego użytku
NL Herbruikbare gezicht-afdekdoeken
PT Drapes laterais reutilizáveis
UA Бараторазове покриття для обличчя

EN The product is intended for professional use only. The reusable face drapes are class I medical devices according to regulation (EU) 2017/745.

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EN INSTRUCTIONS FOR USE

INTENDED USE

The reusable face drapes are intended for use by dental professionals to stop the patient from being covered in airborne splashes and spray during tooth cleaning procedures. The products are reusable.

The products are intended for use by qualified professionals and for external use only. The net material should be placed top of the face as shown in **Figure 1**.

LIMITATIONS FOR USE

Not to be used during surgical procedures, during teeth whitening and other dental treatments. Do not wash with bleach.

REPROCESSING RECOMMENDATIONS

Machine Wash:

- Maximum washing temperature 40°C recommended (**Fig. 2**)
- Duration and dosage refer to disinfectant detergent manufacturer recommendations
- Disinfectant detergent suitable at 40°C. Always read the label and product information before use.

Drying Automated Process:

- Temperature low (**Fig. 3**)

Laundry tips: to maximize the life of your product we recommend to make a first wash with cold water and let them air dry before the first use. This will help you to prevent shrinkage due to washing and drying at too high a temperature.

Number of reprocessing cycles that can normally be tolerated is 20 washes. After each wash the device must be inspected before use: checking for any defects, tears or damage. Device should not be reused when, e.g. signs of material degradation or the maximum number of allowable reuses have been reached.

SHELF LIFE OF PRODUCT

The product has a shelf life of 2 years from date of manufacture (**Fig. 14**). Before initial use, always check that the product is within the stated shelf life. After initial use the drape can be washed a maximum 20 times before product will need to be disposed off.

STORAGE AND HANDLING CONDITIONS

Do not use if package is damaged (**Fig. 12**). Keep away from sunlight (**Fig. 9**). Keep dry (**Fig. 10**) and protected. The device must be inspected before use: any defects, tears or damage also mean the face drape should be discarded.

INCIDENT NOTIFICATION

If any serious incident has occurred in relation to the use of device it should be reported to the manufacturer and the competent authority of the Member State in which the user is established.

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