Medicom SAS
Boulevard de la Chanterie | 49124 Saint Barthélemy d'Anjou
[BP 10059 – 49181 Saint Barthélemy d'Anjou Cedex] France
Tel +33 0 241 963 494 | sales.support.pro@medicom.com | www.medicom.eu.com

Medicom GmbH
Benzstraße 1 | 51381 Leverkusen | Germany
Tel +49 2171 70 66 70 | info.dental.eu@medicom.com | www.medicomeurope.com

Swiss Authorised Representative (CH-REP)
FINAGEST SA, Rue Pierre-Fatio 12, 1204 Genève - Switzerland
Swiss Single Registration Number (CHRN): CHRN-AR-20002401

Instructions for use / Notice d'utilisation / Gebrauchsanweisung / Instrucciones de uso / Istruzioni per l'uso / Instruções de utilização / Gebruiksaanwijzing / Instrukcja użycia / Инструкция за використання / Инструкции За Употреба / Уpute Za Uporabu / Οδηγίες Χρήσης / Návod K Použití / Brugsanvisning / Kasutusjuhend / Användningsanvisning / Käyttöohjeet / Használati Utmutató / Lietošanas Norādījumi / Struzzjonijiet Għall-Użu / Istruțiuni De Utilizare / Návod Na Použitie / Navodila Za Uporabo

Reference code	Product description	Band width	Colour
30045	SafeMatrix™ Band	6.0 mm	Blue
30050	SafeMatrix™ Band	4.5 mm	Green
30055	SafeMatrix™ Contour Band	6.0 mm	Orange
30060	SafeMatrix™ Contour Band	4.5 mm	Yellow
30065	SafeMatrix™ Clear Contour Band	4.5 mm	Pink
30066	SafeMatrix™ Clear Contour Band	6.0 mm	Purple

The device is in direct contact with tissues such as the mucosa, gums and teeth. The device is for single use only. The device is designed to be used only by healthcare professionals trained in its use.

LIMITATIONS FOR USE
Single use only (Fig. 2).

INSTRUCTIONS FOR USE

- 1) Begin by adjusting the SafeMatrix™ band to suit the quadrant in which it will be used. Push the diverter button in on the side where you want to narrow the band. This side must be placed towards the gingiva.
- 2) Place the matrix band around the tooth and carefully guide it into position.
- 3) Tighten the band by turning the thumbscrew clockwise until it fits perfectly. Apply wedges and shape the band using a suitable burning tool if required. Perform restoration using normal clinical procedure.
- 4) Remove the band by turning the thumbscrew counter-clockwise to loosen it. Note: When using single use contoured Mylar® SafeMatrix band ensure polymerisation light can transmit through.

SHELF LIFE OF PRODUCT
The product has a shelf life of 5 years from date of manufacture (Fig. 3). The expiry date (Fig. 5) is indicated on the packaging. Before initial use, always check that the product is within the stated shelf life.

STORAGE AND HANDLING CONDITIONS
SafeMatrix™ bands are single use disposable device and should not be reused. The device must be inspected before use; any defects or damage also means SafeMatrix™ band should be discarded and not used. Do not use if packaging is damaged (Fig. 15). Keep away from sunlight (Fig. 8). Keep dry (Fig. 6).

INCIDENT NOTIFICATION
If any serious incident has occurred in relation to the use of device it should be reported to the manufacturer and the competent authority of the Member State in which the user is established.

DISPOSAL
According to local regulations.


EN INSTRUCTIONS FOR USE

DESCRIPTION AND INDICATIONS
SafeMatrix™ is intended for professional use only. The SafeMatrix™ bands are class I medical devices according to regulation (EU) 2017/745.

INTENDED USE
Made of a plastic holder and a circumferential stainless steel or Mylar band with a tightening thumb screw mechanism. The device is in direct contact with tissues such as the mucosa, gums and teeth. The device is for single use only. The device is designed to be used only by healthcare professionals trained in its use.

LIMITATIONS FOR USE
Single use only (Fig. 2).

INSTRUCTIONS FOR USE

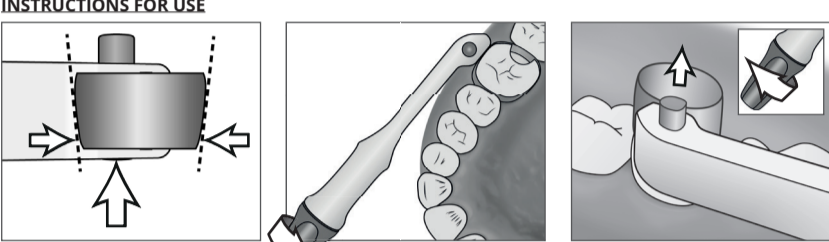- 1) Begin by adjusting the SafeMatrix™ band to suit the quadrant in which it will be used. Push the diverter button in on the side where you want to narrow the band. This side must be placed towards the gingiva.
- 2) Place the matrix band around the tooth and carefully guide it into position.
- 3) Tighten the band by turning the thumbscrew clockwise until it fits perfectly. Apply wedges and shape the band using a suitable burning tool if required. Perform restoration using normal clinical procedure.
- 4) Remove the band by turning the thumbscrew counter-clockwise to loosen it. Note: When using single use contoured Mylar® SafeMatrix band ensure polymerisation light can transmit through.

STORAGE AND HANDLING CONDITIONS
SafeMatrix™ bands are single use disposable device and should not be reused. The device must be inspected before use; any defects or damage also means SafeMatrix™ band should be discarded and not used. Do not use if packaging is damaged (Fig. 15). Keep away from sunlight (Fig. 8). Keep dry (Fig. 6).

INCIDENT NOTIFICATION
If any serious incident has occurred in relation to the use of device it should be reported to the manufacturer and the competent authority of the Member State in which the user is established.

DISPOSAL
According to local regulations.

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